



excibur®

ZARC

	MFD	TAPER	Ø
E20 SMALL	0,95	5%	0,20
E25 REGULAR	1,00	5%	0,25
E35 MEDIUM	1,08	5%	0,35
E45 LARGE	1,15	5%	0,45



KFile
#10



E20
SMALL



E25
REGULAR
85% of cases



E35
MEDIUM



E45
LARGE

500 RPM, 150 CCW, 30 CW
Torque: 4

Glide Path

Shaping & Finishing



Irrigate, recapitulate with a 10 file and re-irrigate after removing each rotary file