


SlimShaper®

ZARC

| | MFD | TAPER | Ø |
|-----|------|-------|----|
| ZS1 | 0,80 | 2-6% | 15 |
| ZS2 | 0,80 | 4-3% | 20 |
| ZS3 | 0,84 | 4-3% | 25 |

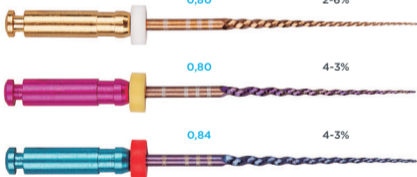

SlimShaper®



KFile
#10

Glide Path



ZS1
RPM: 500
Torque: 3

Shaping



ZS2 **ZS3**
RPM: 500
Torque: 3

Finishing



Irrigate, recapitulate with a 10 file and re-irrigate after removing each rotary file