

# BlueShaper<sup>®</sup>

ZARC



# BlueShaper<sup>®</sup>

ZARC

Z1-Z4 Basic sequence  
(85% of cases)

Z5-Z7 If necessary  
(15% of cases)



**KFile**  
#10

Glide Path



**Z1** **Z2**  
RPM: 500  
Torque: 4

Shaping



**Z3** **Z4** **Z5** **Z6** **Z7**  
RPM: 500  
Torque: 4

Finishing



*Irrigate, recapitulate with a 10 file and re-irrigate after removing each rotary file*