



excalibur®

ZARC

	MFD	TAPER	Ø
<b>E20</b> SMALL	0,95	5%	0,20
<b>E25</b> REGULAR	1,00	5%	0,25
<b>E35</b> MEDIUM	1,08	5%	0,35
<b>E45</b> LARGE	1,15	5%	0,45



**KFile**  
#10



**E20**  
SMALL



**E25**  
REGULAR  
85% of cases



**E35**  
MEDIUM



**E45**  
LARGE

500 RPM, 150 CCW, 30 CW  
Torque: 4

Glide Path

Shaping & Finishing



*Irrigate, recapitulate with a 10 file and re-irrigate after removing each rotary file*