ENDO-BLOCK

Measuring accessory

Importer for the EU:

IPG Dental Group S.L.
C/ Marqués de San Esteban nº8, 1ª A y B,
33206 Gijón, Asturias (Spain)

Shenzhen Denco Medical Co., Ltd

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Embalaje

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ENDO-BLOCK

Before using, please see the instruction manual as indicated below.

Warning

1. Professional use only.
2. Sterilize this product before use.
3. Use this product only for dental service and treatment. Use it in accordance with the intended use.

Precautions and contraindications

No known contraindications.

Product indication

1. Measuring accessory for dental use. Convenient plastic device for setting a precise working time.
2. This measuring device helps to set endodontic stops.

Warning

This Endo-Block is not sterilized. It must be disinfected before use. The temperature should not exceed the established maximum temperature.

Method of use

1. Please disinfect the Endo-Block before using it. Refer to the following sterilization instructions.
2. The sterilizing temperature should not exceed the established maximum temperature.

3. This measuring device helps to set endodontic stops.

4. You can set the stops to confirm the working length of endo devices as shown above.
5. The length of the hole in the upper row depression is 0,5mm longer than that in the right adjacent hole, and the measured length of the hole in the lower row depression is 0,5mm longer than that in the left adjacent hole.
6. When using, the endo files can quickly be put into the hole accordingly, to line up with leveling surface of the measuring table, and adjust down the stoppers to get the working length needed. This way you obtain the working length easy and quick.

7. As shown above, the left text contains manufacturer

information and the right can be used for measuring endo instruments.

8. During using, it can also be used as a rack of endo files.

Sterilization instructions

1. Remove the packaging of the Endo-Block before sterilization. Pay attention to the maximum sterilize temperature of 132°C.
2. If the disinfection solution contains a corrosion inhibitor, it is recommended to wash the instruments before the high pressure steam curing.
3. Place the Endo-Block into de disinfection bag and steam sterilize at 132°C/269.6°F and 210Kpa for 4-10 minutes. The drying time is 20 minutes.
4. Place the block in a sterile packaging and in a dry and clean environment.

Conservation and storage

1. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
2. Do not damage or make pinholes in packaging materials.
3. Do not keep under germicidal lamp to avoid deterioration.

Packaging

Minimum packaging unit: 1pcs/box