

# BlueShaper PRO<sup>®</sup>

ZARC

		TAPER	Ø
Z1		10-2%	0,14
Z2		10-2%	0,17
Z3		5-6%	0,19
Z4		5-6%	0,25

# BlueShaper PRO<sup>®</sup>



**KFile**  
#10

Glide Path



**Z1** **Z2**  
RPM: 500  
Torque: 4

Shaping



**Z3** **Z4**  
RPM: 500  
Torque: 4

Finishing



*Irrigate, recapitulate with  
a 10 file and re-irrigate  
after removing each  
rotary file*