

Gutta-percha points for canals

INSTRUCTIONS FOR USE

Gutta-percha is one of the dental restorative materials. Dentists use it as a root canal filler when treating endodontic diseases. Our gutta-percha got the ShangHai New Excellence Product award.

1. It is white (or pink) spicule, odorless, without irritation with disinfection.
2. It is impervious to X-rays, we clearly see the filling of the canal.
3. It has a certain firmness. It can easily get into the fine and clogged duct.
4. Its diameter and size are in accordance with the duct reamer.

COMPOSITION

High quality natural gutta-percha, zinc oxide.

1) INDICATIONS OF USE

It is used to fill the canal when dentists perform endodontics.

2) CONTRAINDICATIONS

None known.

3) WARNINGS

- It must be disinfected with alcohol before use.
- Should not be applied to allergic patients.

4) SPECIFICATIONS

- It is produced in accordance with ISO 6877. It can be customized in size according to the requirements of the ducts with varieties of types.

Zarc Gutta-percha size 0.04 and 0.06				
Standard specification	.04/25	.04/30	.06/25	.06/30
Length (mm)	28	28	28	28
Diameter tip (mm)	0.25	0.30	0.25	0.30

Zarc Gutta-percha special size

Denomination	X-Fine	Fine
Length (mm)	28	28
Diameter tip (mm)	0.14	0.17

BlueShaper® gutta-percha size

Denomination	Z3	Z4	Z5	Z6	Z7
Length (mm)	28	28	28	28	28
Diameter tip (mm)	0.20	0.25	0.30	0.40	0.50

Excalibur® gutta-percha size

Denomination	E20	E25	E35	E45
Length (mm)	28	28	28	28
Diameter tip (mm)	0.20	0.25	0.35	0.45

SlimShaper® gutta-percha size

Denomination	ZS1	ZS2	ZS3
Length (mm)	28	28	28
Diameter tip (mm)	0.15	0.20	0.25

ApicalShaper® gutta-percha size

Denomination	Z30	Z35	Z40	Z50
Length (mm)	28	28	28	28
Diameter tip (mm)	30	35	40	45



5) STORAGE AND EXPIRY

- Store in a cool place. Keep away from direct sunlight, irritating gases and smoke.
- Store as required.
- The shelf life of gutta-percha is 3 years.

6) PACKAGING

- Box package: 100 pieces/box or 60 pieces/box according to gutta-percha size. The label and signature with proof of conformity on the box.

 Shanghai Xing Yu Medical Equipment Co., Ltd
3 Floor, Building 21, No. 3825 Xin Zhuan Road,
Dong Jing Town, Shanghai. CHINA
hola@zarc4endo.com • www.zarc4endo.com

 Luxus Lebenswelt GmbH
Kochstr. 1
47877 Willich, Germany



Importer for the EU:
IPG Dental Group S.L.
C. Marqués de San Esteban N° 8, A y B
33206 Gijón, Asturias (Spain)