

Absorbent Paper Points

INSTRUCTIONS FOR USE

1. Paper Points are a white spicule and have definite tenacity and intensity. It can easily get into exiguous and bend root-canal.
2. It can strongly sop up tissue saliva and make it dry quickly.
3. Stays straight even when immersed in antimicrobial or disinfectant solution during endodontic treatment.
4. Its diameter sizes are in accordance with root-canal needles.
5. STERILE: It will be irradiated by 25 kGy Gamma rays after packing.

For dental use only.

Choose the same size as dental files and use tweezers to clip the paper points for cleaning the root canal. Single use only.

COMPOSITION

Cotton paper.

1) INDICATIONS OF USE

It is used to absorb fluid from the root canal tissue of the oral cavity when endodontics is performed by dentists.

2) CONTRAINDICATIONS

None known.

3) SPECIFICATION

Zarc Paper Point size 0.04 and 0.06				
Standard specification	.04/25	.04/30	.06/25	.06/30
Length (mm)	28	28	28	28
Diameter tip (mm)	0.25	0.30	0.25	0.30



Zarc Paper Point special size

Denomination	X-Fine	Fine
Length (mm)	28	28
Diameter tip (mm)	0.14	0.17

BlueShaper® Paper Point size

Denomination	Z3	Z4	Z5	Z6	Z7
Length (mm)	28	28	28	28	28
Diameter tip (mm)	0.20	0.25	0.30	0.40	0.50

Excalibur® Paper Point size

Denomination	E20	E25	E35	E45
Length (mm)	28	28	28	28
Diameter tip (mm)	0.20	0.25	0.35	0.45

SlimShaper® Paper Point size

Denomination	ZS1	ZS2	ZS3
Length (mm)	28	28	28
Diameter tip (mm)	0.15	0.20	0.25

ApicalShaper® Paper Point size

Denomination	Z30	Z35	Z40	Z50
Length (mm)	28	28	28	28
Diameter tip (mm)	30	35	40	45

5) STORAGE AND EXPIRY

- Keep below 40°C, airless, in a non-windy and non-corrosive place. The maximum accumulation height is 5 boxes.
- The shelf life of Paper Points is 3 years.

6) PACKAGING


- Box package: 100 pieces/box or 180 pieces/box according to paper point size. The label and signature with proof of conformity on the box.

7) SPECIAL NOTE

- Please stop using it when the color changed (the normal color is white).
- If the benmost pipe is broken, don't use it.
- Single use only.
- Please use it in time after the package has been opened.

8) STERILIZATION

- Store according to request, three years of sterilization term.

 Shanghai Xing Yu Medical Equipment Co., Ltd
3 Floor, Building 21, No. 3825 Xin Zhuan Road,
Dong Jing Town, Shanghai. CHINA
hola@zarc4endo.com • www.zarc4endo.com

 Luxus Lebenswelt GmbH
Kochstr. 1
47877 Willich, Germany



Importer for the EU:
IPG Dental Group S.L.
C. Marqués de San Esteban Nº 8, A y B
33206 Gijón, Asturias (Spain)